PUMPSETS AND ACCESSORIES

Instructions for use | Instrucciones de uso

This page is intentionally left blank

1. Intended use

The Ardo Pumpset should be used in combination with Ardo breastpumps and is intended to be used by lactating women to express and collect milk from their breast.

The Pumpset can be used both as a single pumpset and as a double pumpset and is intended for single users only.

2. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the Pumpset has been correctly assembled and is not damaged.
- Check whether the lip valve is cracked or stuck together. Check whether the smooth side of the flexible membrane pot is facing outwards.
- Check that the adapter tube cover seals well.

Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size fits your nipples.
- Check that the vacuum intensity is not set too high.
- Seek professional help if problems persist.

3. User risks

There are no known contraindications.

Safety regulations: Only use after cleaning (see section on "Cleaning"). After cleaning, check for damage.

4. Technical Information

M	Materials: Polypropylene (PP) / Silicone
	Temperature: operation 41 °F (+5 °C) to 104 °F (+40 °C) / transport -4 °F (-20 °C) to 122 °F (+50 °C)
	Disposal: can be safely disposed of in compliance with local regulations (Non-hazardous waste)
	Packaging: caution, fragile

RH	Relative humidity: 20 - 95 %
P	Air pressure: 700 - 1060 hPa
	Keep packaging away from heat
	Keep packaging away from moisture

5. Product description

Pumpset:

- a.) Adapter tube cover
- b.) Flexible membrane pot
- c.) 26 mm breast shell
- d.) 1 lip valve plus 1 spare valve
- e.) 150 ml bottle
- f.) Tube
- g.) Tube connector
- h.) Bottle cap
- i.) Cap liner

Double Pumpset: two each of a to i

Clinic Pumpset: from a to i

Pumpsetkit: a, b, f and g

WARNING: CHOKING HAZARD – Small parts included. Keep small parts away from babies and children.

6. Accessories

- j.) 36 mm breast shell
- k.) 31 mm breast shell
- l.) 28 mm breast shell insert
- m.) Optiflow massage insert
- n.) Double bottle holder
- o.) 22 mm breast shell insert
- p.) Bottle stand

The two Ardo breast shells j.) and k.) and the breast shell inserts l.) and o.) allow you to select your size individually. The Ardo Optiflow massage insert m.) guarantees an even gentler pumping experience and its massaging action stimulates the flow of milk (can only be used with the 31 mm funnel). The double bottle holder n.) is suitable for use with the Calypso Double Plus breastpump and the bottle stand p.) for Calypso Essentials.


7. Assembling the Pumpset

Assemble as illustrated.

Important: Boil before first use (see section on "Cleaning") and check that the smooth side of the flexible membrane pot is facing outwards.

CAUTION: Let components cool after boiling before touching them.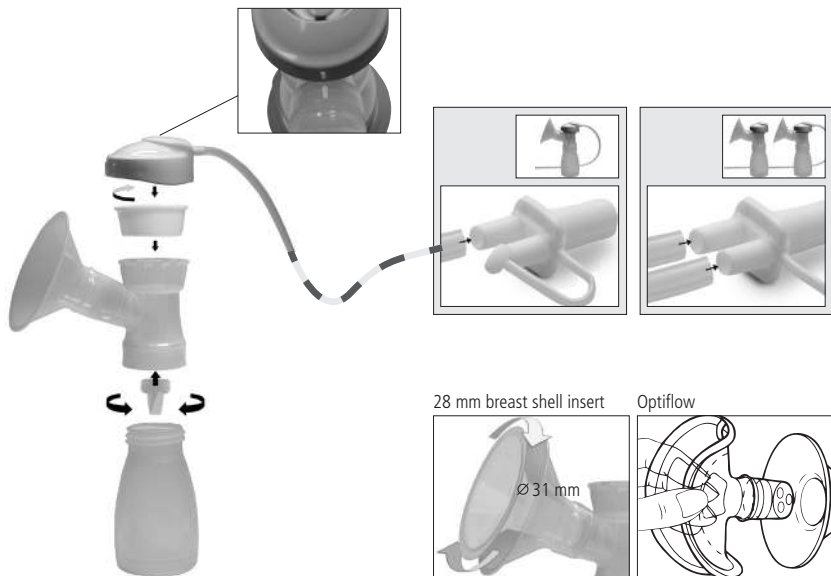

8. Cleaning

Before first use, disassemble and boil all components for 10 minutes. Air dry before assembling and use.

After every use, wash with soapy water and rinse well. Once a day, disassemble and boil the components for 10 minutes or put the components in the dishwasher, then dry and cover with a clean cloth.

The components shown in the illustration (adapter tube cover, tube and tube connector) can, but need not be boiled. It is sufficient to wipe them with a cloth when needed. **Do not clean the lip valve with a brush.**

CAUTION: Let components cool after boiling before touching them.

9. Instructions for use

9.1 Expressing milk with a Pumpset

- Wash your hands thoroughly.
- Attach the Pumpset to the Ardo breastpump.
- Sit down comfortably and relax.
- Centre the nipple so that, when pumping, it moves freely in the breast shell / breast shell insert.
- Switch on the pump.


9.2 Using the double Pumpset

- Attach the second Pumpset to the second opening of the tube connector (see illustration "Assembling").
- Using the Ardo Carum breastpump: Place a Pumpset on your breast, switch on the pump and position the second Pumpset on your other breast.
- Using the Ardo Calypso breastpump: Position both the Pumpsets at the same time (see illustration). Switch on the pump and place one hand on each Pumpset.

10. Choice of breast shell and Optiflow massage insert

If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell / breast shell insert. If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow Massage insert can only be used with the 31 mm breast shell.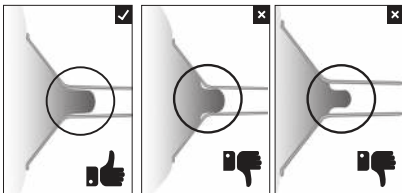

11. Storage of breast milk

Check with your hospital for specific instructions!

At room temperature: 6 hours at max. 77 °F (25 °C); in the refrigerator: 5 days at max. 39 °F (4 °C); in the freezer compartment of the refrigerator (3 °F / -16 °C) 3 - 6 months; in the deep-freezer (< 0 °F / < -18 °C) 6 - 12 months.

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Ardo medical AG