# INSTRUCTIONS FOR USE INSTRUCCIONES DE USO

## **TULIPS** NIPPLE SHIELDS PEZONERAS


Ardo medical AG  
Gewerbstrasse 19  
6314 Unterägeri  
Switzerland  
T +41(0)41 754 70 70  
F +41(0)41 754 70 71  
info@ardo-usa.com  
www.ardo-usa.com


**CE** 0123 Class 2a

Distributed in USA by:  
Comit Developers  
116 Toledo Drive  
Lafayette, LA 70506, USA  
T 844-411-ARDO (2736)  
www.ardo-usa.com  
customer-service@ardo-usa.com

1.)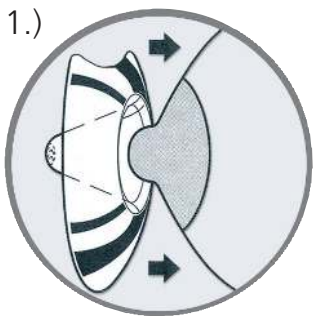

2.)

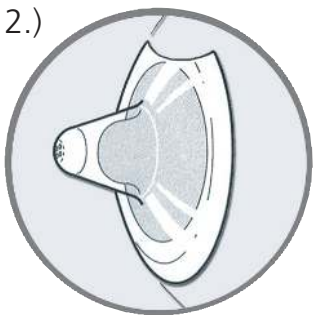## INSTRUCTIONS FOR USE

### TULIPS NIPPLE SHIELDS

#### **Package contains:**

Two silicone nipple shields, size M (20 mm) or L (24 mm), one storage box.

#### **Intended use:**

Tulips nipple shields are suitable for short-term use to help with latching on and to protect sore, sensitive nipples during breastfeeding.

#### **Use:**

1. Moisten the inside of the nipple shield with a little breast milk or water, or use a lanolin nipple cream, so that it adheres more easily to the breast. Pull back the edge of the nipple shield and centre the shield over the nipple. The baby's nose should be at the tulip-shaped cut-out, so that it is touching the mother's skin (image 1).
2. Release the edges of the nipple shield onto the breast (image 2).

#### **Notes:**

Wear only when breastfeeding, remove after use and clean according to the instructions. Frequent sterilisation and boiling causes the material to wear faster. It is recommended to let the nipple shields dry before storing them in the storage box. Stop using the nipple shields if they show any signs of damage. Store the nipple shields and accessories out of the reach of children. Only wear the nipple shields temporarily while breastfeeding, and clean after use according to the instructions. If you have any questions, please contact a healthcare specialist. Tulips nipple shields are latex free.

**WARNING: CHOKING HAZARD – Small parts included. Keep small parts away from babies and children.**

**WARNING: It is important to use a nipple shield under the direction of a healthcare professional or lactation consultant.**

#### **Cleaning:**

Boil the nipple shields in water for ten minutes before the first use, and then once per day thereafter. Clean with soapy water after each use, then rinse well and allow to dry on a clean cloth. The nipple shields and storage box are dishwasher safe.

**CAUTION: Let components cool after boiling before touching them.**

#### **Disposal:**

Harmless, in accordance with local regulations.

Instructions for use in other languages are available at [www.ardomedical.com](http://www.ardomedical.com).

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