
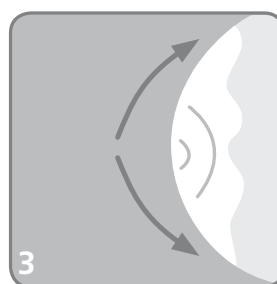
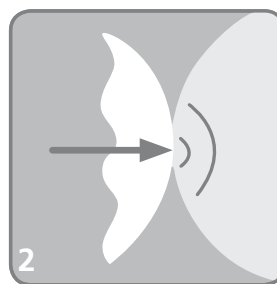
en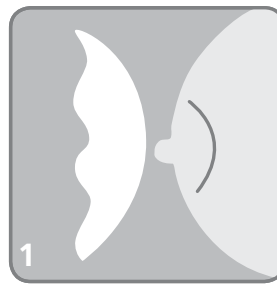

INSTRUCTIONS FOR USE

LILYPADZ RE-USABLE SILICONE BREAST PADS

Packaging contains: Two silicone breast pads, two protective shells, storage case, instructions for use

Indications: Reusable breast pads made from breathable silicone. The pads protect your breasts and clothes from milk leaking, helping you enjoy your freedom. LilyPadz can be used as an alternative to traditional breast pads and are suitable for daily use. They are also ideal for swimming, wearing at night, evenings out and many other occasions. Discreet, reusable, dry.

Contraindications: Do not use if your nipples are sore or if you have thrush (fungal infection). Applying a lot of cream (e.g. Lanolin) to nipples can affect the adhesiveness of the LilyPadz, so only apply cream sparingly. A very strong letdown can also temporarily affect the adhesiveness of the pads. Contact your doctor if you experience irritation or redness on the skin.

Usage:

- Always wash hands thoroughly before use. Breasts must be clean and dry before applying the LilyPadz. When using the LilyPadz for the first time, they should be cleaned as per the instructions.
- Remove the LilyPadz from the protective shells. Push the adhesive side of the LilyPadz (the inner side) outwards.
- Place the LilyPadz centrally over your nipple with the adhesive side against your skin and carefully press down onto your nipple (**important**).
- Smooth out the LilyPadz over your breast and you are ready to go!
- To nurse your baby, remove and clean the LilyPadz before placing them in the protective shells and storing them in the case.

Cleaning instructions: Always wash the LilyPadz after use using hot water and a pH-neutral soap (e.g. a liquid hand wash which does not contain lotions or oils). Every now and then the LilyPadz (without the protective shells) can be boiled (max. 1 minute). **The LilyPadz do not need to be boiled frequently: frequent boiling will shorten the product life!** Do not wash them without using soap. Leave them to dry naturally. Do not place in the dishwasher or microwave. If cared for properly, LilyPadz breast pads can be used for up to two months. Each pair is intended for use by one person only.

- Disposal:** dispose of safely, according to local regulations
- LilyPadz material:** silicone
- Case material:** polycarbonate

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Ardo medical AG